Multilingual Quick Guide

English

Español

Deutsch

Français

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Italiano

Português

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
Fitting

The display can be worn on top of the wrist (Fig. 1) or on the underside of the wrist (Fig. 2)

Fig. 1


Fig. 2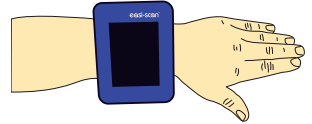


Release the Velcro fastening and slide the straps over your hand (Fig. 3)

Fig. 3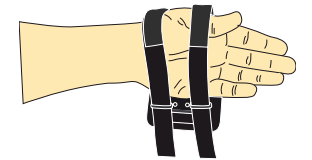


Once the display is comfortably positioned on the wrist, pull the end of the straps to tighten and engage the Velcro fastening to secure (Fig. 4)

Fig. 4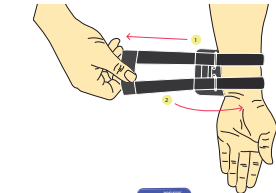

Excess length can be trimmed from the straps. Remove the Velcro tab by peeling it from the end of the straps. Refit the tab at a suitable length and trim any excess using sharp scissors (Fig. 5)

Fig. 5

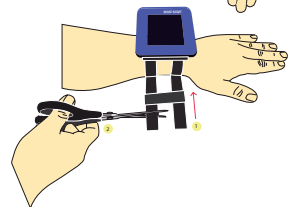Connection to scanner


There are two connectors on the wrist display to accommodate left or right handed use. Use the left connector when using the display on the left wrist and the right connector when using the display on the right wrist. (Fig. 6)

Fig. 6

Make sure that the white dots of the cable connector and wrist display connector are lined up and plug the lead into the wrist display. The connection lead should run naturally from the display up your arm. Plug the other end of the lead into the Easi-Scan Display connector. When the scanner is switched on, the display will automatically power on. Following scanner initialisation the live ultrasound image will be shown on the display. (Fig. 7)

Fig. 7

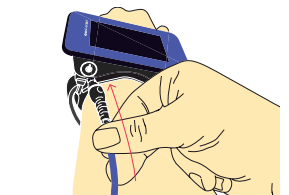Fit the push-fit connector cover into the unused connector to prevent dirt ingress (Fig. 8). Do not connect any other lead to the unused connector.

Fig. 8

Sun hood fitting

The sun hood can be used to improve display visibility in high ambient light conditions.

The sun hood can easily be fitted to the wrist display by stretching the elasticated base over the perimeter of the display. (Fig 9)

Fig. 9


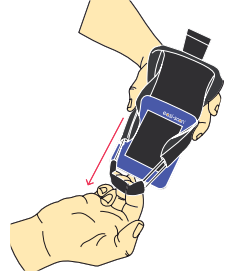
Fig. 10

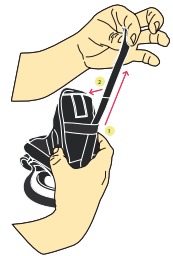The hood is held closed with the elastic strap. To deploy the hood, detach the velcro tab and pull the strap to open the hood. Stretch the elastic strap and secure the velcro tab to keep the hood open. (Fig. 10, 11, 12)

Fig. 11

Fig. 12

When not in use, the hood can be latched closed, and if left attached to the wrist display will give the display screen added protection. (Fig.13)

Fig. 13

Cleaning

Treat the display screen as you would your valued spectacles or sunglasses. Clean immediately after use, before any dirt dries solid. Leave the lead connected while cleaning. Grit and mud should first be removed by rinsing gently to avoid scratching- do not rub the screen. Dry excess water off the housing with a clean soft cloth - do not rub the screen. Clean the screen only with spectacle cleaner fluid and clean cloth or computer screen wipe - allow to evaporate.

The wrist strap can be removed from the display for cleaning.

To attach strap after washing make sure that the plastic rings are at the bottom of the display as it shown on the picture (Fig. 14).

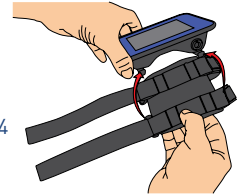
Fig. 14

Slide the strap loops over the ends of all four attachment posts (Fig. 15).

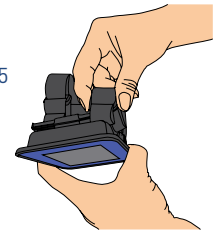
Fig. 15

Care

We have built the wrist display to be rugged and water resistant, it does however contain fragile, sensitive electronics.

The screen is made from high quality optical plastic; care is required to clean this without scratching.

The display Housing is water resistant to withstand use in the rain and gentle rinsing. It will not withstand powerful waterjets (e.g. tap full on or hosing down).The connector plugs and sockets are sealed but if they get wet must be dried out to prevent corrosion.

Specification

Size	120.6mmX91.7mmX40mm (65mm with strap on)
Weight	200 grams (with the hood, excluding connection lead)
Connection	Easi-scan proprietary digital video only
Power	Powered from scanner
Sealing	Splashproof and Dustproof
Operating Temperature	-10 to 40 C
Display Technology	LCD with LED backlight
Display Size (Diagonal)	94mm (3.7")
Display Size	56mm x 75mm
Picture format	Portrait view
Pixel resolution	480 x 640
Luminance	300 cd/m2
Contrast ratio	500:1

Certification

BCF Wrist Display is compliant with all CE requirements (for use with BCF Easi-Scan only). As an accessory that can only be used with Easi-Scan. BCF wrist display is covered by the Easi-Scan Compliance certificate. It is ROHS and WEEE compliant.

Note: All BCF Technology products are to be used for animal applications only.

UK +44 (0)1506 460 023

North America +1 (507)529 8200

IRE +353 (0)42 932 0070

Australasia +61 (0)3 98948980

Distributors contact details, video manuals can be found at www.bcftechnology.com.

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© BCF Technology Aug 2015